



USA & Canada (800) 421-1587 & (800) 392-0123
(760) 438-7000 - Toll Free FAX (800) 468-1340
www.linearcorp.com

PRODUCT BULLETIN
PB2011-011
ISSUE DATE 12/23/2011

Important Product Safety Announcement

The following important product safety Warning Addendum is now being included with all Linear Personal Emergency Reporting System (PERS) products.

We strongly encourage you to distribute a copy of the Warning Addendum to every person who has been provided any Linear PERS product, including transmitters, receivers, and consoles.

Printed copies of the Warning Addendum are available at no cost. Should you wish to receive printed copies of the Warning Addendum for distribution to your PERS subscribers, please contact your Linear Customer Service representative at 800-421-1587 and ask for Linear Warning Addendum part number 232933.

WARNING

TO REDUCE THE RISK OF SEVERE INJURY OR DEATH TO PERSONS, REVIEW THIS IMPORTANT SAFETY INFORMATION PRIOR TO USE

Linear LLC is issuing this Warning to all subscribers who use a Personal Emergency Reporting System (“PERS”).

Linear manufactures and sells a number of different PERS models that consist of a console or receiver unit and one or more transmitters. PERS transmitters may be worn on a wristband, with a belt or pocket clip or as a pendant. The pendant style transmitters are equipped with a neck cord or “lanyard” so that the transmitter can be worn around the neck. **IF YOU WEAR THE LANYARD AROUND YOUR NECK AND IT BECOMES ACCIDENTALLY ENTANGLED, IT MAY PRESENT A CHOKING OR STRANGULATION HAZARD THAT COULD RESULT IN SERIOUS INJURY OR DEATH TO YOU.** There have been several incidents involving lanyards that have resulted in serious injury or death.

PERS are most commonly used by elderly subscribers and others who may have limited mobility, and this risk applies especially to subscribers who may use wheelchairs, walkers, beds with guard rails, or similar objects with protrusions on which a transmitter lanyard could become entangled.

Because PERS transmitters can be worn on a bracelet, clip or as a pendant, Linear urges users to consult with their caregivers or service provider to determine which style is most appropriate for them. It is important for users to consider each style before making an informed choice regarding the most suitable method for wearing their transmitter.

To report any issues related to your PERS and/or any of its components, please do not hesitate to contact Linear at 1-800-421-1587